



ORFIGUARD

HIGH PRECISION PROTECTION MASK

Protection of the nose,
nose bridge, and eye sockets



The
personal fit



Orfiguard is a one-size-fits-all protection mask. The set includes one thermoplastic mask, straps, and a strip of Luxofoam.

- 1 Place the strip of Luxofoam on your patient's nose bridge.
- 2 Activate the mask in a water bath or oven, and mold it on your patient. Let the material harden.
- 3 If required, easily adjust areas at the nose and eyes by trimming with scissors or by spot-heating pressure points.
- 4 Remove the strip of Luxofoam and fasten the straps. Fit the mask one last time.

Your athlete can resume play with protection.



**The
personal fit**

www.orfit.com



ORFIT INDUSTRIES

Vosveld 9A | B-2110 Wijnegem | Belgium
T (+32) (0)3 326 20 26
welcome@orfit.com

ORFIT INDUSTRIES AMERICA

810 Ford Drive | Norfolk, VA 23523
T 516-935-8500
welcome@orfit.com

26-08-2022
Art. N° 52025

